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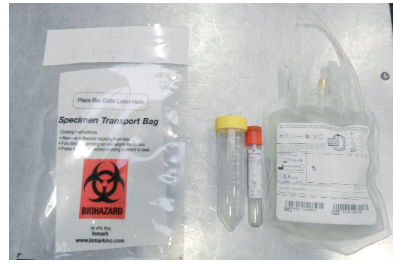
Once the collection is complete place all samples (Cord tissue, maternal blood and Cord blood) inside the Specimen Transport Bag or bags (if required). Seal the bag by removing the tear off strip and folding over the flap.

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Once the samples are inside the Specimen Transport Bags and sealed, remove the Gel Pack from the shipping container and wrap it around the sealed bag(s)

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Place the Gel Pack wrapped samples back inside the shipping Container. Place the polystyrene lid securely on top of the shipper.

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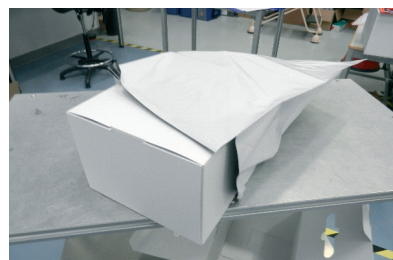
Place the lid back on the Shipping Container and leave any accompanying documentation on top of the polystyrene container.

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Close the Outer Shipping Box and attach the security seal. Peel off backing and secure to front of the courier box.

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Place the sealed box inside the shipping pouch which is provided and seal. The sample is now ready for shipment to the Processing Laboratory. Keep the box at room temperature. Please do not refrigerate.

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**It is important these instructions are carried out correctly to ensure the samples are kept at optimum temperatures during transit. Thank You.**