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1

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
1

Place the placenta in a clean bowl.


Clean and sterilise the umbilical cord using the enclosed alcohol swab.

OPTIONAL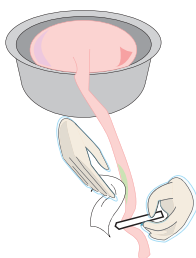


2

Close the blue clamp at the cannula that is not being used. Twist and remove the protective cap on the other cannula and insert it in the artery on the umbilical cord. Tip the blood bag to mix the umbilical cord blood with the anti-coagulant liquid in the blood bag.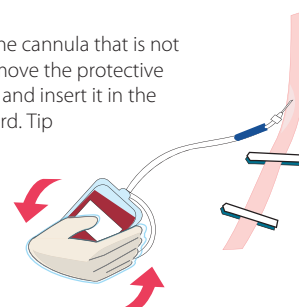


3

The blood supply to the umbilical cord can be increased by tipping the placenta and positioning the blood bag lower than the inserted cannula.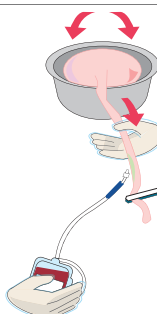


4

If the cannula/artery becomes blocked, sterilise a new area closer to the placenta with a new swab and insert the blood bag's spare cannula into the artery.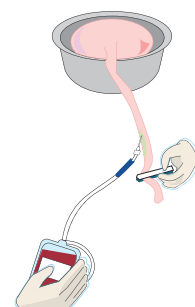


5

Once all blood has been drained from the placenta, squeeze the blood in the tube into the blood bag with the help of the blue clamp.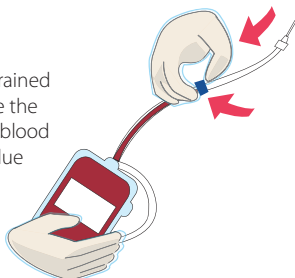

6

Close the tube with the supplied white umbilical cord clamp approx. 5 cm above the blood bag. Cut off the tube with cannulas and discard in a suitable container.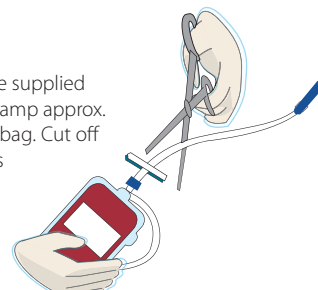

7

Stick a barcode label from the Collection Kit on top of the existing label on the Blood Bag.

8

Put the labeled Blood Bag together with the absorbant material in the Plastic Bag with air-tight closure. Then place it between the 2 thermo-gel Bags, so that the temperature measurement chip is in contact with the Plastic Bag containing the Blood Bag.

9

With the elastic band (from the Collection Kit) tie together the 2 thermo-gel Bags and the air-tight Plastic Bag (which includes the Blood Bag) and then place them into the styrofoam box in the Collection Kit. Close the styrofoam box.

10

Fill in the required data on the "Collector Form" and sign it on back page (only by the medical personal). Place it in the Collection Kit.

11

Close the Collection Kit by using the provided transparent sticker. Fill in the following data on the outside of the Collection Kit: Hospital name and address and Mother's name.