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## INSTRUCTIONS FOR COLLECTING CORD TISSUE

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### PROCEDURE

#### Step 1 Preparing for the collection

- Once the baby has been born and the cord blood collected, open the cord tissue kit. Ensure that the label on the 50ml tube is signed and dated.
- Using the gloves provided, or your own if preferred, open the saline ampoule and pour the contents into the 50ml tube, replace the lid of the tube.

#### Step 2 Preparing the cord tissue

- The cord tissue should be prepared for shipping as soon as the cord blood has been collected.
- Identify a section of cord away from any puncture sites and as straight as possible.
- Initially clean the section of the cord using the sterile gauzes to remove as much blood as possible from inside and outside the cord tissue, always wipe in the same direction each time.
- Clean the cord again using the IPA wipes, wiping the cord in the same direction.

#### Step 3 Collecting the cord tissue

- Cut approx. a 15cm section of the cord that has been cleaned; as a guide the 50ml tube is approximately 12cm long.
- Place the cord tissue into the 50ml tube and close the lid tightly taking care not to cross-thread it. Invert the tube (lid down) and ensure there is no leakage.

#### Step 4 Preparing for transit

- Place the 50ml tube into the Biohazard Specimen Bag with the absorbent pad and then seal this bag.
- Sign the Proof of Collection form to state that cord tissue has been collected.
- Place the Biohazard Specimen Bag in the shipping box along with the umbilical cord blood sample.